

Month/Year:

Serial Number:

SPIFE® Touch Preventive Maintenance Schedule

For more information on correct procedures, read the product inserts contained in every box of gels.

Daily Items, If Used	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Clean electrophoresis chamber																																
Clean electrodes																																
Clean rigid antisera template																																
Clean sample base																																
Perform maintenance wash																																
Weekly Items																																
Clean stain level detectors																																
Monthly Items																																
Clean waste level sensor																																
As Needed																																
Replace fuse(s)																																
Replace contact sheet																																
Clean gel holder																																
Clean antisera tray																																
Clean instrument																																
Replace vat(s) if mold develops																																
With Each Reagent Refill																																
Rinse vat(s)																																

Note: Duplicate this page for maintenance copies. Initial each item as required. Refer to section ten, Maintenance, of the Operator's Manual for details. If you are uncertain about how to perform any step, please contact Helena Laboratories Electronic Customer Service Dept. (1-800-231-5663) for more information.
Helena Laboratories D7000008A, 10/15