

Month/Year: _____ Serial Number: _____

Table 10.1 SPIFE® Nexus Preventive Maintenance Schedule

After every test	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Clean Reagent Roller 10.9.2																															
Clean visible spills 10.9.3																															
Clean Blotter Holder (if used) 10.3.4																															
Daily Items, If Used	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Prime fluid system 10.9.6																															
Clean Platen Cover electrodes 10.9.8																															
Clean top of Dryer & Dryer Cover 10.9.10																															
Clean Gel Cassette 10.9.1																															
General inspection for cleanliness																															
Inspect platen contact sheet 10.9.5.2																															
Weekly Items	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Clean bottom track of scanner (see Addendum)																															
Monthly Items	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Sanitize Wash/ Destain vat & tubing 10.9.14																															
Bleach fluid & waste containers 10.9.12																															
Replace Antisera Spreader Tip (call Service)																															
As Needed	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Backup parameters 10.9.21																															
Replace platen contact sheet 10.9.17																															
Sanitize pipette tubing (call Service)																															
Clean Reagent/ Sample Tray Stations 10.9.7																															
Clean optical fluid sensors 10.9.15-16																															
Clean bottom surface of Dryer (see Addendum)																															

Note: Duplicate this page for maintenance copies. Initial each item as required. Refer to section ten, Maintenance, of the Operator's Manual for details. If uncertain about how to perform any step, please contact Helena Laboratories Electronic Customer Service Dept. (1-800-231-5663) for more information.
 D7000011A, 06/23