



H-PETTE® with Blade Instructions For Use

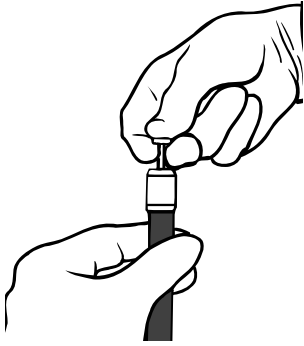

1. Place H-Pette's cannula in the center of the stopper of the collection tube. Gently press until the cannula completely penetrates the septum. The tube is now ready for use. **Caution: cannula is sharp; care should be taken in handling.**

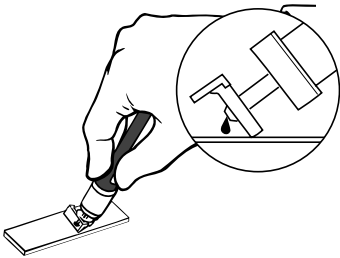

2. Invert tube at an angle with paddle side up as shown. Gently press the round edge of the H-Pette to the glass slide to transfer sample to the slide.



3. Grasping the circular disk as shown, rotate tube to turn paddle side down. Use H-Pette's plastic blade just as you would a glass slide. A 15-25° angle of incidence gives optimum results. H-Pette can be removed with the H-Pette Remover tool according to the instructions provided, or disposed with the collection tube.

Cat. No.	Item
3611	H-Pette without Blade, 1000 Pieces
3612	H-Pette with Blade, 1000 pieces
3613	H-Pette Remover
3615	H-Pette II without Blade, 1000 Pieces

For Sales, Technical and Order Information, and Service Assistance, call 800-231-5663 toll free.

H-Pette manufactured under U.S. Patent Number 5,163,583; other patents pending.

Helena Laboratories warrants its products to meet our published specifications and to be free from defects in materials and workmanship. Helena's liability under this contract or otherwise shall be limited to replacement or refund of any amount not to exceed the purchase price attributable to the goods as to which such claim is made. These alternatives shall be buyer's exclusive remedies.

In no case will Helena Laboratories be liable for consequential damages even if Helena has been advised as to the possibility of such damages.

The foregoing warranties are in lieu of all warranties expressed or implied including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Helena Laboratories
Working for you

Beaumont, TX