

Month/Year:	Serial Number:																														
SPIFE Nexus Preventive Maintenance Schedule																															
After every test	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Clean Reagent Roller 10.10.2																															
Clean Visible Spills and General Inspection for Cleanliness 10.10.3																															
Clean Blotter Holder (if used) 10.10.4																															
Daily Items, If Used	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Clean Gel Cassette 10.10.1																															
Inspect platen contact sheet 10.10.5																															
Prime fluid system 10.10.6																															
Clean Platen Cover electrodes 10.10.8																															
Weekly Items	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Clean Scanner 10.10.12																															
Clean Antisera Spreader Tip Slot D3000280																															
Monthly Items	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Maintenance Wash 10.2																															
Replace Antisera Spreader Tip 10.10.9																															
Bleach Fluid Bottles 10.10.10																															
Quality Control	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
NDDC and OD Step Tablet 8.3																															
As Needed	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Inspect Flow 10.1																															
Backup parameters 10.6																															
Bleach Pipette Tubing (call Service)																															
Clean Reagent/ Sample Tray Stations 10.10.7																															
Replace Contact Sheet (call Service)																															
Clean Pipette Wash Station 10.10.11																															
Clean Optical Fluid Sensors 10.10.13																															
Clean Waste Level Sensor 10.10.14																															
Replace Carbon Electrode Insert 10.10.16																															

Note: Duplicate this page for maintenance copies. Initial each item as required. Refer to section ten, Maintenance, of the Operator's Manual for details. If uncertain about how to perform any step, please contact Helena Laboratories Electronic Customer Service Dept. (1-800-231-5663) for more information.